

# Medical-D


## REprogramming of D-mode and SMILE-mode

This instruction is applicable for Medical-D camera's with the D- and SMILE-dail mode

In some situations, the programmed D-mode and SMILE-mode may be lost, for example when this camera is not used for longer time or when batteries are not charged timely. The steps below enable you to RE-programm the Dial Mode. KEEP THIS CARD WELL ARCHIVED. For regular functions on this camera see also the digital instruction manual of SONY;

You can download it via this link: [http://rd1.sony.net/help/dsc/1815/h\\_zz](http://rd1.sony.net/help/dsc/1815/h_zz)

- 01 Controlwheel
- 02 Dial Mode
- 03 MENU-button
- 04 Fn (Function)-button
- 05 Shutter (+ Zoom)
- 06 OK / Enter button

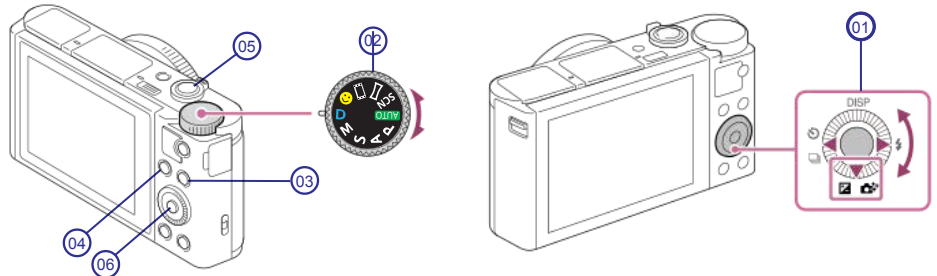

### INITIALIZING

MENU > > Setup7 > Setting Reset > OK > Initialize > Enter > OK > Enter > OK  
> Choose Language > Set Area / Date / Time

### PROGRAMMING

- 1 Go to Shutter > select Zoom > '100 mm'
- 2 Set the Dial Mode on 'M'
- 3 Push MENU-button > turn Controlwheel while changing functions and go to menu >
  - 1/4 AF (Focus Area) > select > 'Center'
  - 1/5 (Exposure) > ISO > select > 'ISO AUTO'
  - (Exposure) > Metering Mode > select > 'Center'
  - 1/7 (Color/WB/Img.Processing) > go to > White Balance > Flash > select > 'A-B: B1.5'
  - 2/5 (Display/Autoreview) > go to > Gridline > select > 'Diag. + Square Grid'
  - (Auto Review) > select > 2 secPress MENU-button to leave the menu
- 4 Turn Controlwheel > select Shuttertime > '1/60'
- 5 Press 1 x 'down' on Controlwheel > > Rotate aperture to > 'F8.0'
- 6 1/3 Go to Memory > 1 / 2 > select > 'Memory'  
> select with Controlwheel > 1 > 'OK' > leave the menu
- 7 Go to Shutter > select Zoom > '70 mm'
- 8 Set the Dial Mode on 'A'
- 9 1/7 (Color/WB/Img.Processing) > go to > White Balance > AWB > 'OK'  
Creative Style > Portret > Leave the menu
- 10 Turn Controlwheel > select > 'F6.3'
- 11 1/3 Go to Memory > 1 / 2 > select > 'Memory'  
> Select with Controlwheel > 2 > 'OK'  
> leave the menu

Medical-D is a registered Trademark

The Medical-D camera is designed & produced by DHM-dental B.V.

version: Aug 2021  
file: TD2.2\_ProgramCart dutch-english MK2)\_8-2021-D-SMILE-mode

MDR:2017/745

first class performance

**DHM**  
DENTAL

DHM-dental B.V.  
Simon Stevinweg 48  
6827 BT ARNHEM  
Netherlands