  
RELIANCE ORTHODONTIC PRODUCTS, INC.  
1540 W. Thorndale Avenue,  
Itasca, IL 60143 USA  
1-800-323-4348 Made in USA | Rx Only  
RelianceOrthodontics.com

 RelianceOrthodontics.com

 ITC TRANSLATIONS  
ITC USA  
2000 PGA Blvd., Suite 4440  
Palm Beach Gardens, FL 33408

EC REP MD

EMERGO EUROPE  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

AUSTRALIAN SPONSOR  
EMERGO AUSTRALIA  
Level 20, Tower II, Darling Park, 201 Sussex St, Sydney,  
NSW 2000, Australia

U.K. PERSON RESPONSIBLE  
EMERGO CONSULTING (UK) LIMITED  
c/o Cr360 – UL International  
Compass House, Vision Park Histon  
Cambridge  
CB24 9BZ  
England, United Kingdom

SWITZERLAND AUTHORIZED REPRESENTATIVE  
MedEnvoy Global BV  
Leidschendam-Voorburg, Zug Branch Office  
Gotthardstrasse 28, 6302 Zug, Switzerland

**RESTORE™**  
**DIAMOND**  
**POLISHING PASTE**

INSTRUCTIONS FOR USE

 V3 OCT.2023  
1639

 **RELIANCE®**  
ORTHODONTIC  
PRODUCTS

SCAN FOR MORE INFORMATION

## EN - ENGLISH

**INDICATIONS FOR USE:** Restore™ diamond polishing paste is intended for use in restoring an aesthetic polish on porcelain, zirconia or ceramic surfaces following orthodontic treatment.

**INSTRUCTIONS FOR USE:**

- Rinse the zirconia, ceramic or porcelain surface with water.
- Apply Restore™ liberally to the surface.
- Polish with a Prophy cup or soft buff.
- Rinse with water.

Repeat process above to enhance luster.

**TEMPERATURE LIMITATION:** Store at room temperature (15-25°C)

**CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPS BEFORE EACH PATIENT.**

◊ Avoid contact with skin and eyes.

Protective eyewear should be worn by the patient and dental staff.

**PATIENT POPULATION:** The intended patient population is pediatric and adults requiring orthodontic treatment as indicated by clinicians.

**CONTRAINDICATIONS:** Bonding attachments to patients with compromised oral health, such as poor oral hygiene, periodontal disease, or preexisting dental conditions (i.e., decalcification, carious lesions, large restorations) can increase the risk of tooth damage.

**SAFE DISPOSAL:** Dispose of contents and container to hazardous or special waste collection point, in accordance with local, regional, national and/or international regulation, a licensed hazardous-waste disposal contractor or collection site except for empty clean containers which can be disposed of as non-hazardous waste, a hazardous or special waste collection point.

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from use of or suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**RX ONLY:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**NOTICE:** Report any serious incident to Reliance and the competent authority of the state in which the user and/or patient is established.

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