 Pressure Moulding Technique

BIOPLAST® XTREME Kit

Herstellanleitung / Instruction manual /
Mode d'emploi / Manual de instrucciones

SCHEU

Materialliste

BIOPLAST® XTREME 5,0 mm
BIOPLAST®-Isoliermittel
Heiz-Support

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BIOPLAST® XTREME 5,0 mm
BIOPLAST® agent isolant
Support de chauffe

List of materials

BIOPLAST® XTREME 5.0 mm
BIOPLAST® insulating agent
Heating Support

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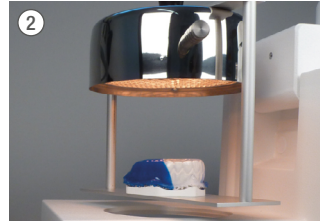
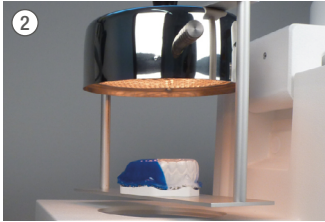
BIOPLAST[®] XTREME Instruction manual

Apply one thin layer of BIOPLAST[®] insulating agent only to those parts of the working model which will be covered by the mouthguard. Do not allow the insulating agent to dry out. Embed model in pellets with the clear V-shaped part of the BIOPLAST[®] XTREME covering the desired extension of the anterior section (eg. from 3-3). Place pellet cover on pellets. Make sure the plate is fixed with the soft side down and the labels being placed in the direction of the heater. The imprint on the label has to be readable. Heat the plate by setting the code or recommended heating times according to the instructions. Lock pressure chamber and open after cooling phase has finished.

Warm up the occlusal surfaces under the infrared heater using the heating support.

Apply BIOPLAST® insulating agent on the antagonist model and put the models into the articulator for counter bite. Cool down with compressed air or water and remove. Polish the edges using the finishing set. If necessary, polish thermally.

Cut mouthguard to the desired shape. Polish the edges with OSAMU-Polisher or DIMO®PRO wheel.

Please note:

To be used only for the above indication and by qualified personnel. For single use. Serious incidents in connection with the use of the product must be reported immediately to the manufacturer and the competent authority!

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SCHEU-DENTAL GmbH